GERMOCID[®] ENZIMATICO

Detergente concentrato

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Concentrated detergent

for instruments
with proteolytic enzymes

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EN - Properties

GERMOCID ENZIMATICO is a proteolytic enzymatic detergent for washing and cleansing of the medical and dental instruments. It is efficacious also in presence of organic residues, even dry ones, and on persistent dirt: it can be used on glass, plastic and metal materials. For **PROFESSIONAL USE**, suitable for manual cleaning and also for use in ultrasound equipment. **Instructions for use:** Dilute **GERMOCID ENZIMATICO** at 0.6-10% in a liter of tap water, possibly warm, and mix thoroughly. Before dipping the instruments in the prepared solution, it is advised to decontaminate them with a specific product and rinse with cold water, then dip them in the solution of **GERMOCID ENZIMATICO** for 10 minutes; in presence of dry residues or persistent dirt extend immersion time to 1 hour, and if necessary use a brush.

Composition: Non-ionic surfactants 15-30%; anionic surfactants 5-15%; enzymes; perfume (Limonene, Citral), preservative (Benzisothiazolinone). **Warnings:** Causes serious eye damage. Causes skin irritation. Harmful to aquatic life with long lasting effects. Wash hands thoroughly after handling. Avoid release to the environment. Wear protective gloves / eye protection / face protection. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a **POISON CENTER / doctor**. Contains: Alkaline fatty alcohols. For professional use. Store in a cool and dry place, away from heat sources. After opening, if properly stored, the product keeps the validity indicated on the label.

EU - Complete translation on www.germa.it

BG	Концентриран мени препарат за инструменти с протеолитични ензими	LV	Koncentrēta šķīduma šķīdums instrumentu ar proteolītiskiem enzīmiem
CS	Konzentrovāný detergent pro nástroje s proteolytickými enzymy	NL	Geconcentreerd reinigingsmiddel voor instrumenten met proteolytische enzymen
DA	Konzentreret rengøringsmiddel til instrumenter med proteolytiske enzymer	PL	Koncentrowany środek do czyszczenia narzędzi i instrumentów proteolitycznymi enzymami
DE	Konzentrierter Reiniger für Geräte mit proteolytischen Enzymen	PT	Detergente concentrado para instrumentos com enzimas proteolíticas
EL	Συγκραματικό καθαριστικό εργαλείων για πρωτοκόλλο καθαρισμού με πρωτεολυτικά ένζυμα	RO	Detergent concentrat pentru instrumente cu enzime proteolitice
ES	Detergente concentrado para instrumentos con enzimas proteolíticas	RU	Концентрированный очиститель для инструментов с протеолитическими энзимами
ET	Proteolüütiliste ensüümidega kontsentreeritud pesuvahend	SK	Konzentrovány čistiaci prostriedok pre nástroje s proteolytickými enzýmami
FR	Déodorant concentré d'instruments à enzymes protéolytiques	SL	Konzentrirano daljinsko sredstvo za instrumente s proteolitičnimi encimi
HR	Konzentrirani deteržent za instrumente s proteolitičnim enzimom	SR	Konzentrovani deteržent za čišćenje instrumenata sa proteolitičnim enzimom
HU	Proteolitikus enzimokat tartalmazó koncentráltum műszerek tisztítószer	TR	Aküler (en proteolitik enzimler içerir) konsantre deterjan
LT	Konzentruotas valdikis įrankiams su proteolitiniais fermentais	AR	مطهر مركزى للأدوية (يحتوي على إنزيمات بروتوليتية)

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