

EN

**Ready-to-use disinfectant  
for invasive medical devices  
detergent with quaternary ammonium salts**

**ATTENTION**

**Properties**

GERMOCID 2% is a ready-to-use solution for cleansing, disinfection and decontamination of invasive medical devices before sterilization. Broad spectrum product, GERMOCID 2% performs a rapid and effective cleansing and disinfectant action against bacteria, fungi, mycobacteria and viruses (Adenovirus, HIV-HBV-HCV). Practical and easy to use, GERMOCID 2% is not corrosive, is compatible with the normal components used in medical surgeries and increases its action in ultrasonic tanks. The high efficacy and rapidity of GERMOCID 2% both as a disinfectant and as a detergent in the presence of organic material (blood - saliva - organic exudates) allows it to be used on all medical, dental, surgical and aesthetic instruments. The removal of dirt does not require mechanical action

**Instructions for use:** Ready to use. Soak the instruments to be treated in the solution according to the degree of disinfection and contact time needed. Rinse thoroughly with water before use. The solution maintains its efficacy up to 24 hours. Contact times: 15 minutes vs Bacteria and Fungi; 60 minutes vs Mycobacteria and Viruses.

**Composition:** 100.0 g of product contain: Benzalkonium chloride 2.0 g; O-phenylphenol 0.2 g; co-formulants; depurated water q.s. to 100.0 g.

**Warnings:** H319 Causes serious eye irritation. H315 Causes skin irritation. H400 Very toxic to aquatic life. H411 Toxic to aquatic life with long lasting effects. P273 Avoid release to the environment. P391 Collect spillage. P280 Wear protective gloves /eye protection /face protection. P337-P313 If eye irritation persists, get medical advice/attention. Shake well before use. The product is incompatible with detergents and anionic surfactants.

Store in a cool, dry place, away from heat.

After opening, if properly stored, the product keeps the validity indicated on the label. Do not use damaged packaging.

For professional use.

**Detergent in compliance with EC Regulation No. 648/2004**

**Validity:** 3 anni

**Batch number and expiry date:** see the bottle

**MEDICAL DEVICE CE 0546 European Directive 93/42/ECC**

**Manufacturer:**

GERMO S.p.A. - Via Giotto 19/21 - 20032 Cormano (MI) Tel. 02 663 01 938

[www.germo.it](http://www.germo.it) - [info@germodis.com](mailto:info@germodis.com)

MADE IN ITALY

*The warnings may change with updates to the product safety data sheet*